


CHEIROSCOPE SET UP INSTRUCTIONS

Needed Parts:


- 1 Tiltboard Kit
- 1 Tiltfoot
- 1 Flipper clip
- (+4.00 or +5.00) Flipper
- 1 Cheiroscope Module (which includes this subset of parts):

- 1 Cheiroscope Tiltfoot
- 1 Mirror Mount
- 1 Blue Stick
- 1 Mirror
- 1 Magnet
- Cheiroscope Charts (not pictured)

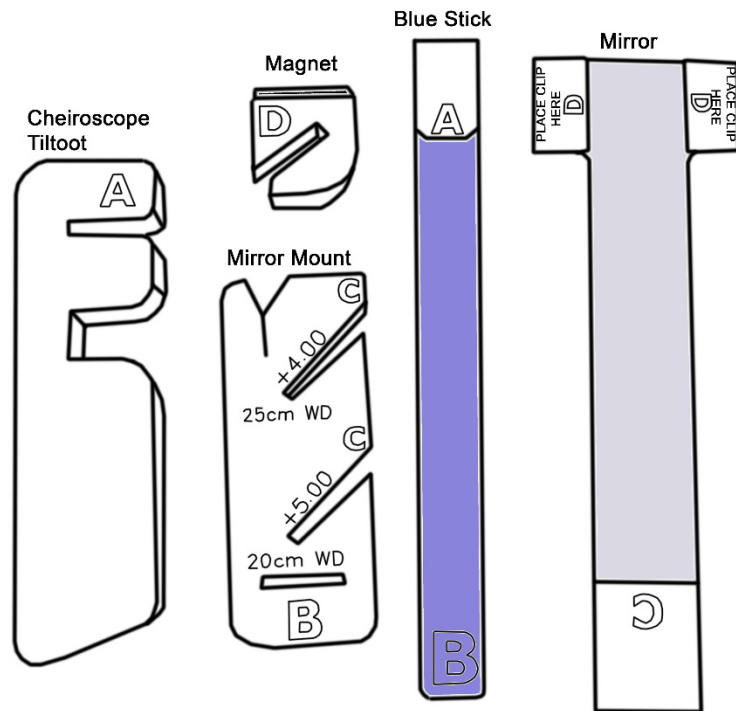


Instructions:


1. Turn the tiltboard kit so that the whiteboard side is facing up and the blue book rest is toward you. Then place a tiltfoot (not the cheiroscope tilt foot) on the right side of the rear of the kit.

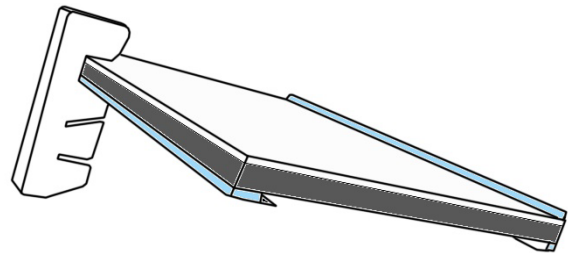


2. Place the cheiroscope tiltfoot on the left rear of the tiltboard kit.

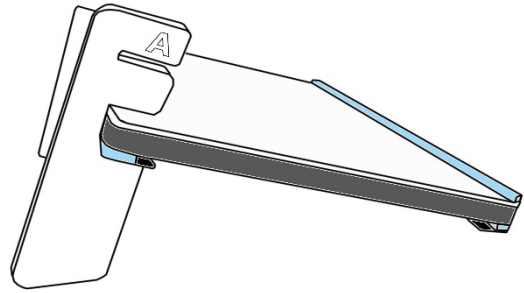


3. Insert the foam side "A" of the blue stick firmly into the "A" slot in the cheiroscope tiltfoot.

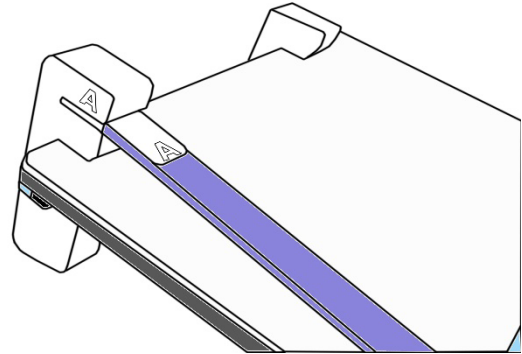


4. Orient the mirror mount vertically with the lettering oriented upright and the flat side of the mirror mount facing the left. Slide the "B" slot of the mirror mount over the "B" end of the blue stick. Slide the mirror mount to approximately half the length of the blue stick.

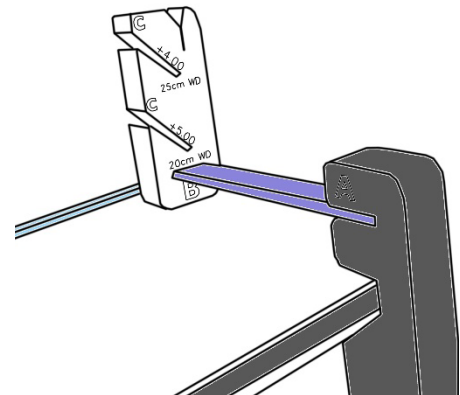


5. Insert the "C" end of the mirror, with the mirror side facing up, into either the +4.00 or +5.00 "C" slots of the mirror mount according to your activity's needs.

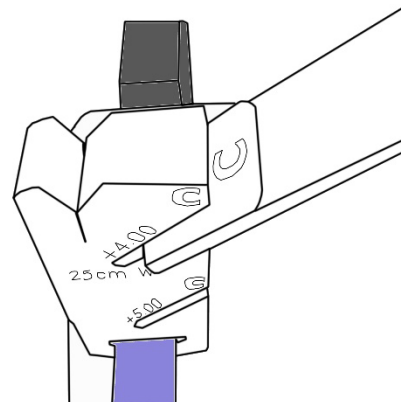

6. Insert magnet onto the foam end of mirror with the magnet facing up.

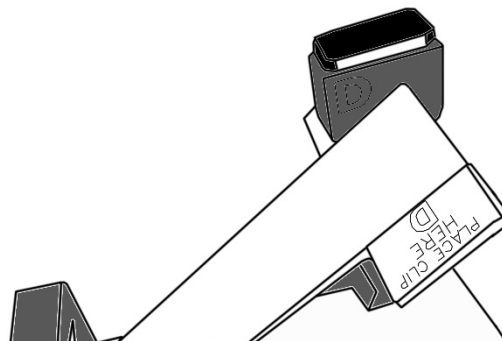

7. Insert the charts into the vertical slot at the top of part mirror mount. *Note: charts stay more rigid if you place 2-4 charts in the slot at a time.

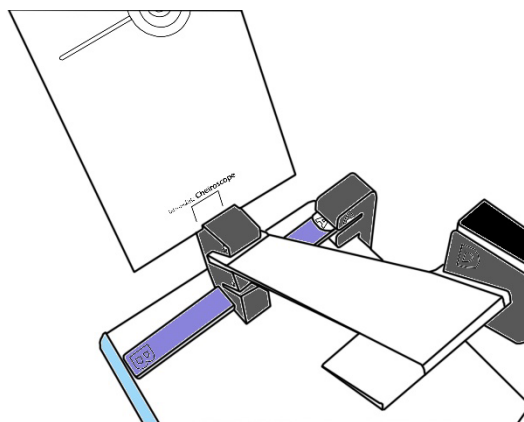

8. Once you have secured the flipper clip on the (+4.00 or +5.00) flipper place the flipper clip on the magnet.

